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50 ml
Sterile solution

EN

What the Protective Preparation for Bladder Mucosa is (HYDEAL CYST)

HYDEAL CYST is a sterile solution for temporary vesical application in a syringe pre-filled to 50 ml with Luer – Lock Adapter.

HYDEAL CYST solution is instilled into the bladder and should be used in conditions where it is necessary to restore and/or protect the layer of glycosaminoglycans (GAGs) which constitute the mucosal layer that lines and protects the vesical urothelium.

Description

HYDEAL CYST is a ready-made single use solution for intravesical instillation.

Every 50 ml syringe of HYDEAL CYST solution contains Hyaluronic acid sodium salt (1.8%) and Hydeal-D (0.2%).

The inner lining of the bladder (urothelium) comprises a layer of GAGs, in other words complex macromolecules (including Hyaluronic Acid), which forms a protective mucosal barrier that makes the inner wall of the bladder impermeable to urine by acting as a barrier against the infectious agents and toxic substances it contains (e.g. bacteria, microcrystals, proteins, ionic and non-ionic residues).

A (qualitative and quantitative) alteration in the GAG layer of the urothelium results in reduced barrier function: this allows the urine to penetrate the inside of the bladder wall, causing painful inflammation and irritation (such as cystitis of various etiology).

HYDEAL CYST is a solution for vesical instillation made with highly concentrated Hyaluronic Acid and Hydeal-D, an ester of hyaluronic acid.

Hyaluronic Acid is recommended in conditions where it is necessary to restore and/or protect the GAG layer. HYDEAL CYST, due to its long-lasting action, offers excellent adhesion to bladder mucosa, helping restore the protective barrier.

Indications

HYDEAL CYST is indicated as an intravesical treatment to restore/protect the GAGs of the urothelium in cases where loss of same can result in frequent and recurrent conditions (such as interstitial cystitis, bladder pain syndrome, haemorrhagic cystitis, cystitis caused by infections, trauma, urolithiasis, urinary retention, and cystitis induced by radiation or chemotherapy). The treatment with sodium salt of Hyaluronic Acid and HYDEAL-D, reconstitutes the physiological protective barrier function of the bladder wall against toxic urinary irritants, reducing irritation, inflammation and helping to alleviate pain.

Method of use

Wash your hands well before applying and proceed as follows:

- In sterile conditions, insert a bladder catheter fitted with a Luer-Lock connector into the bladder
- Remove residual urine
- Remove the syringe from the pouch by tearing where indicated

- Remove the cap from the pre-filled syringe and attach to the catheter
- Instil the 50 ml solution of protective preparation into the empty bladder
- Remove the catheter once the instillation is complete

The protective solution should be retained in the bladder for as long as possible, from a minimum of 30 minutes up to 2 hours.

To alleviate the symptoms associated with this condition, it is advisable to instil the protective solution into the bladder once a week for 4 weeks. Then, based on the attending doctor's opinion only, less frequent administering is permitted – bi-monthly or monthly – until the patient's symptoms are resolved.

The attending doctor should supervise any prophylactic use of HYDEAL CYST.

Warnings

HYDEAL CYST is advised for use in suitably hygienic environments and by healthcare staff that are suitably informed regarding bladder instillation treatment. Product functionality is only guaranteed by the integrity of the primary and secondary packaging (syringe and pouch).

Do not administer to patients with known hypersensitivity reactions. Discontinue use if adverse reactions occur and consult your doctor if symptoms persist. As clinical evidence on the use of Hyaluronic Acid in babies, pregnant women and breast-feeding women is not available, treatment with this protective preparation in these patients is not recommended

Read the instructions for use carefully

Single use product

use the entire contents of the pre-filled syringe for each individual application

Sterile product

If the solution is reused, sterility is no longer guaranteed and can become a vehicle for infection.

Do not dispose of empty containers into the environment. Dispose of in accordance with current legislation.

KEEP OUT OF THE REACH OF CHILDREN. NOT FOR PARENTERAL ADMINISTRATION.

Storage

Store at +5° to +30° C

Fidia Farmaceutici S.p.A.
Via Ponte della Fabbrica 3/A
35031 Abano Terme (PD), Italia

Distributeur
Memidis Pharma bv Benelux
Kerkenbos 1077-R, 6546 BB Nijmegen, The Netherlands

Rev.A - 07/09/21