
Manual

Erection System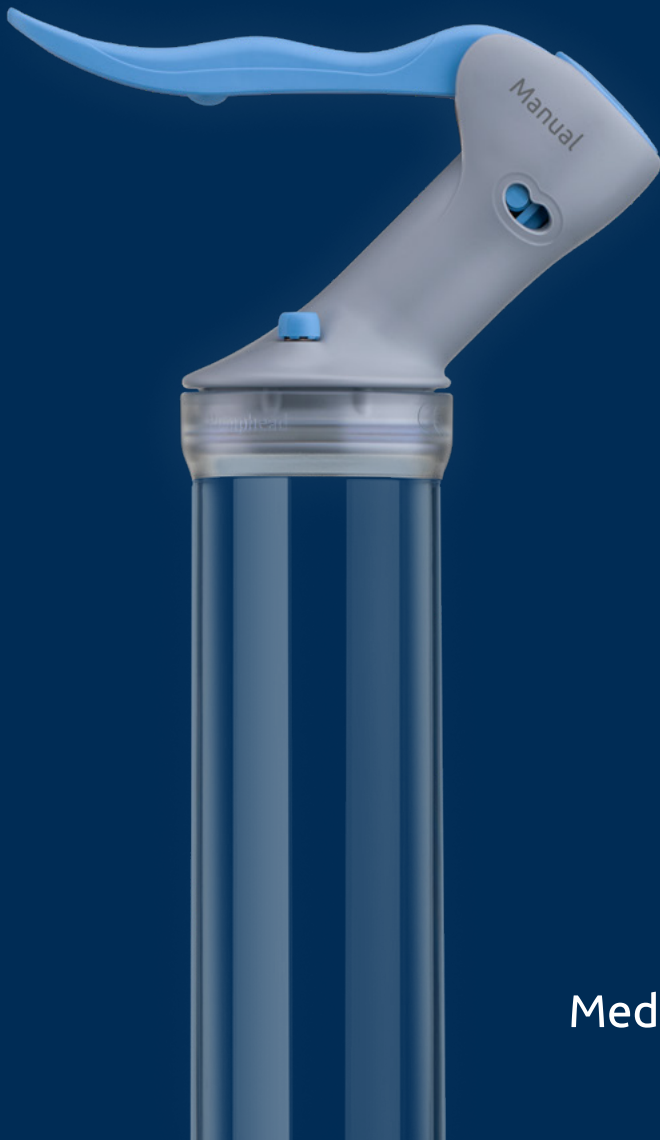

Deutsch
English
Français
Español
Italiano
Português
Nederlands
Dansk
Suomi
Norsk
Svenska
Polski
Čeština
Türkçe
ελληνικά
Slovenčina
Български
Русский
العربية
עברית

Medintim

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Dear customer,
Thank you for choosing our product.

Read all of the instructions for use carefully before using the product for the first time, keep them for later use and follow the instructions.

Contents

- 1. General information
- 2. Application
 - 2.1 How to prepare the application
 - 2.2 How to use the erection aid
 - 2.3 How to select the proper constriction ring
- 3. Cleaning and storing the device
- 4. Side effects / contraindications
- 5. Disposal
- 6. Warranty
- 7. What material are the parts made of?
- 8. Spare and wearing parts
- 9. Complaints

Delivery contents

The Manual Erection System comprises the following parts:

- 1 pump head
- 1 transparent cylinder
- 1 large grey ring loader
- 1 small grey ring loader
- 1 grey cone
- 5 constriction rings in differing sizes
- 1 tube of lubricant
- 1 set of instructions
- 1 bag

Please note the shelf life of the included constriction rings and the lubricant.

1. General information

When the penis is stiff, this is called an erection. If a man has problems getting an erection, we speak of erectile dysfunction. The Manual Erection System is a device that helps you to get your penis stiff. Therefore, in this booklet, the Manual Erection System is also referred to as an erection aid.

What are the causes of erectile dysfunction?

A variety of causes can lead to erectile dysfunction. The most common ones include diabetes, thyroid disease, liver or kidney disease, high blood pressure treatment and cardiovascular disease. Prostate surgery often also leads to erectile dysfunction. Depression or exhaustion can also lead to erectile dysfunction. Sometimes, erectile dysfunction is a side effect of medication, e.g. drugs against high blood pressure. Smoking or drinking too much alcohol is another possible cause.

How does the erection aid work?

The pump head sucks the air around the penis out of the cylinder. This creates a negative pressure (vacuum). This causes the blood vessels in the penis to fill with blood, and the penis stiffens. The constriction ring prevents the blood from flowing out. After removing the constriction ring, the blood flows back and the penis becomes soft.

2. Application

2.1 How to prepare the application

All product-related illustrations can be found on the cover flap of this booklet.

Check the device to ensure the packaging is free from external damage and complete. Make sure before use that the device and accessories are visibly undamaged and that all packaging material has been removed.

- Lay out all parts ready (Fig. 1)
- Press the pump head onto the larger opening of the cylinder, as shown in Fig. 2
- First, attach the large ring loader on the other side of the cylinder and then the smaller ring loader onto it (Fig. 3)
- Insert the cone into the small ring loader (Fig. 4)
- Apply some lubricant to the tip of the cone (Fig. 5)
- First, place the largest constriction ring on the top of the cone. Slide the constriction ring over the cone onto the small ring loader. Now take off the cone (Fig. 6)
- The erection aid is now ready for use.

2.2 How to use the erection aid

The erection aid is for your personal use only. It may not be passed on to third parties, nor be used by them. Never use this erection aid if you have drunk alcohol or have taken recreational drugs. Do not continue using the erection aid if it does not work properly.

Important: Never wear the constriction ring on the penis for more than 30 minutes. Do not fall asleep with the constriction ring on the penis! Use the device only in your domestic environment, and not under the shower or in the bathtub.

- Remove the pubic hair directly on the penis and around the root of the penis.
- The device must fit the skin tightly.
- Apply some lubricant to the edge of the ring loader. (Fig. 7)

- Introduce the penis through the ring loader into the cylinder. (Fig. 8)
- Press the device tightly to your body.
- Pull the blue lever on the pump head. Thus, you pump air out of the cylinder. (Fig. 8)

Do you see a good erection in the cylinder?

- Then push the constriction ring from the ring loader onto the stiff penis.
- Slide the constriction ring quickly and as far as possible down to the base of the penis. (Fig. 9)
- The "air" button on the pump head is marked with a dot. (Fig. 10)
- When you press the "air" button, the cylinder is bled (i.e. air is let in, reducing the vacuum).
- This allows air to enter the cylinder, and you can easily detach the erection aid from the body.
- The constriction ring on the penis causes the penis to remain rigid. (Fig. 11)
- Now you can have sex.

After sex:

- Briefly pull the constriction ring apart on the flaps.
- Wait until the blood has flown back from the penis into the body.
- Then pull the constriction ring off the penis. When you use the erection aid, you will have no visible ejaculation upon orgasm (climax). (Fig. 12)
- The constriction ring retains the semen (ejaculate) in the penis.
- As soon as you remove the constriction ring, the seminal fluid flows out of the penis.
- The temporary retention of the semen in the penis is not harmful.
- After using the erection aid, clean it as described in section "How to clean the erection aid"
- Dry the parts.
- Put all parts back into the bag and keep the erection aid out of the reach of children.

Application notes:

Application of this erection aid is easy. Nevertheless, it is better to try the erection aid initially alone

and unhurried. Then you can show the erection aid to your partner. Once you both know how to use the erection aid correctly, you can enjoy sex again. Even at an advanced age, in illness or physical limitation.

2.3 How to select the proper constriction ring

To keep your erection as long as possible, you need to choose the proper constriction ring. If the constriction ring is too large, your penis will not remain stiff for long. If the constriction ring is too tight, it may cause pain. In case of pain, take off the constriction ring and try a larger constriction ring. When you use the erection aid for the first time, it is best to start with the largest constriction ring. If the constriction ring is too large, try a smaller one until you find the right size for yourself. The erection aid includes 2 ring loaders. The smaller one is slightly narrower, the larger one a bit wider. If the smaller ring loader is too narrow, just use the larger one. The two ring loaders are suitable for all constriction rings. Thus, it does not matter which constriction ring size you are using. If both ring loaders are too narrow, you can slip the constriction ring directly onto the cylinder. The erection aid also includes the cone. It helps you to slip the constriction ring more easily onto the ring loaders. Alternatively, you can slip the constriction ring manually onto the ring loaders without using the cone. Use only the supplied constriction rings. From time to time, the constriction rings need to be replaced. You can make repeat purchases from your dealer.

3. Cleaning and storing the device

Before cleaning, take the pump head off the erection aid. The pump head may only be cleaned with a damp cloth from the outside. The pump head must never be held under water as liquid can otherwise enter and damage the pump head. Do not use any detergents or solvents. All other parts can be easily cleaned with warm water and mild soap. Store the erection aid in the pouch provided in order to avoid contamination.

4. Side effects / contraindications

If you have a blood coagulation or blood formation disorder (e.g. sickle cell anaemia, blood cancer, bone marrow tumours, anticoagulant therapy), contact your doctor before you first use the product.

You can get a bruise (haematoma) at the base or at the top of the penis. Sometimes, there may even be slight tissue injury, when the penis was not erect for a long time. If you have diseases that cause a prolonged erection or in which sexual intercourse should be avoided, do not use the erection aid. If you have a cardiovascular disease, you should ask your doctor before you first use the product in order to clarify possible risks.

If you are not sure whether you can apply the erection aid without any problem for your health, ask your doctor or health care professional.

5. Disposal

Repair and disposal information:

You must not repair the device yourself. Proper functioning will then no longer be guaranteed.

The pump head must not be opened.

In the interest of the environment, the pump head must not be disposed of with household waste at the end of its service life. Disposal may be arranged at suitable collection points in your country. Dispose of the pump head in accordance with the WEEE Guideline (Waste Electrical and Electronic Equipment). In case of questions, please contact the authority responsible for disposal.

6. Warranty

Almost all parts have a 2-year warranty, as prescribed by applicable law.

The warranty does not apply:

- in case of damage resulting from incorrect use.
- for wearing parts such as constriction rings
- in the case of the customer's own fault
- once the device has been opened

You should send in the erection aid, if possible, every 3 years for technical inspection. The serial number can be found on the underside of the pump head.

7. What material are the parts made of?

- Cone: Plastic (PP)
- Small ring loader: Biocompatible thermoplastic elastomer (TPE)
- Large ring loader: Plastic (PP)
- Cylinder: Plastic (polycarbonate)
- Constriction rings: Biocompatible thermoplastic elastomer (TPE)
- Bag: Nylon®

8. Spare and wearing parts

Use only original accessory parts.

For charging, use only the charging cable and the USB mains components supplied. You can order the replacement and wearing parts from the service address. To get the current reordering list, please contact service@medintim.de or your dealer.

9. Complaints

In case of complaints or questions regarding our service, please contact the following:

Phone: +49 6105- 20 37 20
Email: service@medintim.de
www.medintim.de

If we ask you to send the defective product, please send the product to the following address.

KESSEL Medintim GmbH
Nordendstr. 82–84
64546 Mörfelden-Walldorf
Germany

Outside Germany, please contact the dealer where you purchased the device.

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