HYALOFEMME®

EN | with Hydeal-D® 0.2%
Paraben free formulation

Product information
The vaginal epithelium is soft and supple thanks to a delicately balanced mechanism that maintains the correct degree of hydration and lubrication.
At certain times during a woman's lifetime (e.g. after childbirth, during breastfeeding or in menopause), following particular pharmacological treatments or at times of increased stress, this delicate balance may be upset, causing considerable discomfort characterised by vaginal dryness and a burning or itching sensation.
Burning and irritation are often associated with tiny lesions to the epithelium, caused by lack of lubrication.
In such cases, it is advisable to use a local product that favours hydration of the mucosa and helps to restore natural suppleness and softness.
HYALOFEMME is a colourless gel with strong hydrating properties that contains Hydeal-D®, a derivative of hyaluronic acid that allows a prolonged release on the application site.
Hyaluronic acid is a natural molecule responsible for maintaining a correct level of hydration in the tissues, besides playing a fundamental role in tissue repair processes.
The formulation based on Hydeal-D® offers an optimal adhesion to the vaginal mucosa, has a prolonged hydrating action and favours the natural healing of microlesions.

Composition
Principal component: Hydeal-D® 0.2% (prolonged-release hyaluronic acid derivative)
Other components: Propylene glycol, Methylpropanediol, Carbomer, 1,2-Hexanediol, Caprylyl glycol, Sodium hydroxide, Purified water.
The formulation contains neither parabens nor other preservatives.

Indications
Gel complementary to treatment for vaginal dryness of various origin; aid in the natural healing process of friction-induced microlesions in the vaginal mucosa.

- How to use**
1. Screw the applicator firmly onto the tube nozzle
 2. Squeeze the tube to fill the applicator until the plunger stops.
 3. Unscrew the applicator from the tube. It is advisable to expel a small amount of gel from the applicator before use, so that the tip is lubricated before being introduced into the vagina.
 4. After thoroughly washing your hands and the area around your vagina, insert the applicator into the vagina while assuming a crouching or supine (lying down) position and push the plunger until all the gel has been expelled.
 5. After use, extract the applicator and push the plunger in order to expel it from the applicator. Rinse away any residual gel from the components, clean with an antiseptic solution and then rinse thoroughly. Drain away any drops of water. Insert the plunger back into the cannula.
 6. Keep the applicator in the box until next use.

Frequency of use
The frequency with which the product should be used depends on how dry the vaginal mucosa is. One application every three days for a period of thirty days is recommended, unless otherwise prescribed. The treatment does not require interruptions during menstruation.

Contra-indications
Proven individual hypersensitivity towards the product.

Warnings and Precautions
In cases of vaginal infection, consult your doctor before using the product.
In patients with particular individual sensitivity to any component of the formulation and/or at the beginning of treatment, transient vaginal burning or heat sensation may occur. In case of persistent burning, it is advisable to discontinue the treatment and to consult your doctor.
The vaginal applicator should be used by one patient only in order to reduce the risk of cross contamination.
In case of pregnancy, breastfeeding and use in children, consult your doctor before using the product since the safety and efficacy of HYALOFEMME in those cases have not been established.
The product does not influence the results of gynaecological tests.
This product is compatible with lubricated/ non-lubricated latex and polyisoprene condoms.
This product is not compatible with polyurethane condoms.
Do not use concomitantly with products containing quaternary ammonium salts, as benzalconium chloride, because hyaluronic acid can precipitate in their presence.
If the packaging presents damages that compromise the integrity of the tube, do not use the product.
Do not discard the empty packaging in the environment.
Do not use the product after the expiry date indicated on the packaging.
Keep out of the sight and reach of children.

Undesirable effects
As for other products used for the treatment of vaginal dryness, small whitish and odourless discharges might occur.

Storage
Store at a temperature not exceeding between 2°C and 30°C.
Do not freeze.

Presentation
Box containing one 30g tube and one vaginal applicator sealed in an envelope.

Manufacturer
Fidia farmaceutici S.p.A.
Via Ponte della Fabbrica 3/A
35031 Abano Terme (PD) – Italy

Date of the latest revision of the instructions for use: September 2019

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